Erkoloc-pro, transparent
blu • green • pink

DIN EN ISO
13485 : 2010
9001 : 2008

Erkoform-3/3d/3dmotion
Erkopress 300 Tp/-ci

1,0 mm, 160 °C
1,3 mm, 160 °C
2,0 mm, 170 °C
3,0 mm, 165 °C
4,0 mm, 160 °C
5,0 mm, 160 °C

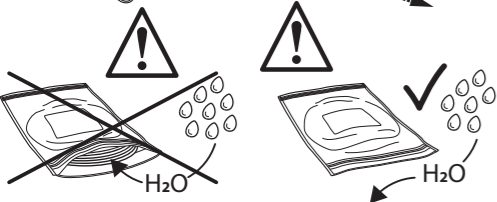
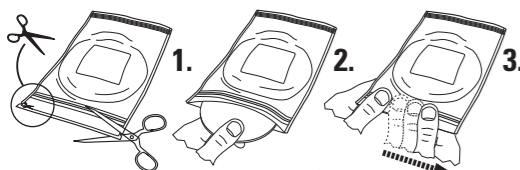
1,0 mm, 2:00 min
1,3 mm, 2:00 min
2,0 mm, 2:00 min
3,0 mm, 3:00 min
4,0 mm, 4:00 min
5,0 mm, 5:00 min

Erkoform-RVE
Erkopress ES 200 E

1,0 mm, 1:00 min
1,3 mm, 1:10 min
2,0 mm, 1:55 min
3,0 mm, 2:40 min
4,0 mm, 3:25 min
5,0 mm, 4:30 min

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EN Erkoloc-pro, transparent • blu • green • pink

Chemical characteristics: PETG, copolyester, hard side • TPU, polyurethane, soft side • PE, insulating foil: polyethylene
Harmless to health, BgVV (Federal Health Office) listed thermoplasts with tested biocompatibility.

Allergic reactions are improbable but cannot be excluded.
Insoluble in water, inactive, harmless to ground water.

Disposal/Recycling: General recycling for plastic/household waste.

Field of use: (see brochure thermoforming technique and material card) All kinds of comfortable occlusal and bruxism splints, Silensor-sl. Adjustment is possible with the Occluform (-3), by grinding or by addition.

Material properties: Durable, resistant double layer plate, hard/soft. Hard layer adjustable by addition using acrylics (Resilit-S, 817 501 + 817 503).

Attention, sensitive to humidity: Immediately after having taken a plate reclose the aluminium bag and keep it closed.

Finishing: (see brochure thermoforming technique) Finishing set Quick 2 (110 877). The insulating foil on the soft layer will only be removed after thermoforming and finishing.

Plastification: Place the foil in the devices in such a way that the spacer/insulating foil is pointing towards the model. For machines without control: Test softness of foil with an instrument. If permanent impressions result, thermoform.

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