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EG-Richtlinie 93/42/EWG

Beiliegende Dokumente beachten

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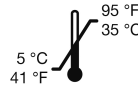
**Lagerung:**


Trocken lagern

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Empfohlene Lagertemperatur

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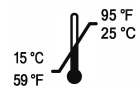
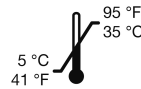
Recommended storage temperature

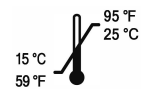Recommended working temperature

Mouldable, condensation cured silicone material for the fabrication of orthosis directly on the patient. Easy to grind, tear resistant, elastic, comfortable to wear after setting.

**Application:**

For the fabrication of pressure relief orthosis, ball shells, toe supports and interdigital toe wedges directly on the patient.

- Mould the mass without adding hardener paste to determine the necessary quantity.
- Mix Erkoton-W 30 – 45 sec. with hardener paste until a uniform mixed colour is achieved.
- After mixing, form Erkoton-W to a ball.

The patient must stand during the hardening process of **Erkoton-W**. **Erkoton-W** can be combined with **Sitran-OF** even after the hardening process. In case involving clavus or similar conditions, no hole should be provided so that the humid milieu will encourage the healing process. Cover open spots or injuries of the foot always with sterile wound cover foil.

**Dosage:**

To 1 level measuring spoon of **Erkoton-W**, 2 cm of hardener paste are added.

**Technical data:**

- mixing time 30 – 45 sec.
- total working time 1.5 – 2 min.
- setting time 6 – 8 min.
- approx. 45 Shore A
- mixing can be carried out manually or by machine
- more hardener will accelerate, less hardener will delay setting
- galvanisation is not recommended
- disinfection of the impression with usual material is possible
- washable

**Attention:**

**Avoid eye contact with the hardener paste; avoid stains on clothes.**

The user of our products is the only person responsible for the use and the application of our products. Our verbal and written advice has to be considered as advice without obligation also regarding possible protection rights of third parties. The user is not exempt from the obligation to check our products personally concerning suitability for the intended application. The perfect quality of our products is guaranteed according to our General Terms and Conditions. In any case and for all damage our liability is limited to the value of the goods supplied by us.

(manufacturer)

Erkodent Erich Kopp GmbH • Siemensstraße 3 • 72285 Pfalzgrafenweiler, Germany  
Tel. + 49 (0) 74 45/85 01-0 • Fax + 49 (0) 74 45/85 01-15 • info@erkodent.com • www.erkodent.com

**EN ISO • 13485:2016 • ISO 9001:2015**

**ERKODENT®**