Erkoton 20/30

45 °C
113 °F
5 °C

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Erkoton 20: Shorehärte 20 Shore A / **Erkoton 30:** Shorehärte 30 Shore A

EN: Instructions for use

Intended use: Silicone for the fabrication of orthopaedic impressions, pressure protection and relief in case of toe malpositioning.

Field of application: Erkoton is exclusively suitable for the fabrication of impressions as well as pressure protection and correction orthoses in podiatry, orthopaedic technology and shoe technology. Manufacture, see instructions, www.erkodent.com  

Contraindications: In case of already known allergic reaction to silicones, a specific allergy to Erkoton should be excluded.

Precautions: Erkoton is recommended for the above use only. Observe storage instructions.

Caution: Use is limited to the fabrication of orthopaedic impressions, custom pressure protection and reliefs. Use is under the responsibility of the practitioner. Use only on the recommendation or prescription of an authorised practitioner. For single use only. Allergic reactions are unlikely but cannot be excluded. Incorrect application may result in the material not curing properly and the stated properties not being achieved. Serious incidents in connection with Erkoton must be reported by the user to the competent authority and to the manufacturer.

Erkoton 20: Shore hardness 20 Shore A / **Erkoton 30:** Shore hardness 30 Shore A

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