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Erkoton 20/30

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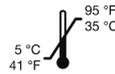
recommended storage temperature

recommended working temperature

Erkoton 20/30, mouldable, addition cured silicone material, highly elastic, tear resistant, easy to grind, two Shore A hardnesses available, comfortable to wear.

Application:

For the construction of pressure reliefs, orthoses, interdigital toe wedges or toe compensation. One level measuring spoon (transparent) of hardener mass is mixed with one level spoon (orange) of **Erkoton 20/30**. Mix quickly and thoroughly within 30 sec. until a uniform colour is achieved. Mixing the material for a longer time will lead to air bubbles and folds which will reduce the durability of the orthosis considerably. After mixing, the material should be formed to a ball or strip and moulded at the foot. Modelling should be finished within 2 min. to avoid - as already mentioned above - air bubbles and folds. The patient should be asked to stand during the forming of the orthosis. **Erkoton 20/30** can be ground by abrasive belts, pumicers or multiple-discs. To polish the surface we recommend to use **Lisko** polishing discs fine, white (art. no. 413 678).

Dosage:

To 1 level measuring spoon of **Erkoton 20/30**, 1 level measuring spoon of hardener mass is added. Proportions of mixture 1 : 1.

Technical Data:

- mixing time 30 - 45 sec.
- total working time 2 - 2.5 min.
- setting time 6 - 8 min.
- app. 20 or 30 Shore A
- type 0 - very high consistency - mouldable (acc. to DIN EN ISO 4823:2015)
- disinfection of the orthosis is possible
- washable

Important:

Close container immediately after use! Use coloured spoon and marked lid only with marked container! Do not bring in contact with latex gloves, disinfectant, foot cream or something like that.

The user of our products is the only person responsible for the use and the application of our products. Our verbal and written advice has to be considered as advice without obligation also regarding possible protection rights of third parties. The user is not exempt from the obligation to check our products personally concerning suitability for the intended application. The perfect quality of our products is guaranteed according to our General Terms and Conditions. In any case and for all damage our liability is limited to the value of the goods supplied by us.

(manufacturer)

Erkodent Erich Kopp GmbH • Siemensstraße 3 • 72285 Pfalzgrafenweiler, Germany
Tel. + 49 (0) 74 45/85 01-0 • Fax + 49 (0) 74 45/85 01-15 • info@erkodent.com • www.erkodent.com

EN ISO • 13485:2016 • ISO 9001:2015

ERKODENT®