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EG-Richtlinie 93/42/EWG

Beiliegende Dokumente beachten

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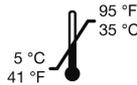
Lagerung:


Trocken lagern

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Empfohlene Lagertemperatur

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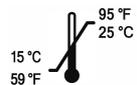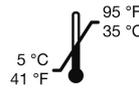
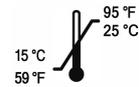
Recommended working temperature

Dosage:

To 1 level measuring spoon of **Kneton**, 1.5 cm of hardener paste are added.

Application:

For the fabrication of impressions directly on the patient for correction orthosis, forefoot and toe compensation, nail clasps or artificial nail plates.

- Mix **Kneton** 30-45 sec. with hardener paste until a uniform mixed colour is achieved.
- After mixing, form **Kneton** to a ball and form it to the patient

Cover open spots or injuries of the foot always with sterile wound cover foil. The patient must stay during the hardening process of Kneton, if necessary, the heel of the foot must be supported. **Kneton** can be combined with **Sitran-OF** even after the hardening process.

With the fine impression material **Sitran-OF**, an even more exact impression can be produced, the so-called double impression.

Hint:

Higher room temperatures and more hardener will accelerate the settings.

Attention:

Avoid eye contact with the hardener; avoid stains on clothes.

Technical data:

- mixing time 30 – 45 sec.
- total working time 1 - 1.5 min.
- remouvable after 3 – 5 min.
- app. 60 Shore A
- mixing can be carried out manually ir by machine
- more hardener will accelerate, less hardener will delay setting
- disinfection of the impression with usual material is possible

The user of our products is the only person responsible for the use and the application of our products. Our verbal and written advice has to be considered as advice without obligation also regarding possible protection rights of third parties. The user is not exempt from the obligation to check our products personally concerning suitability for the intended application. The perfect quality of our products is guaranteed according to our General Terms and Conditions. In any case and for all damage our liability is limited to the value of the goods supplied by us.

(manufacturer)

nt Erich Kopp GmbH • Siemensstraße 3 • 72285 Pfalzgrafenweiler, Germany
Tel. + 49 (0) 74 45/85 01-0 • Fax + 49 (0) 74 45/85 01-15 • info@erkodent.com • www.erkodent.com

EN ISO • 13485:2016 • ISO 9001:2015