

Lisko-Polierscheiben fein, weiß



Lisko-Polierscheiben haben eine offene Struktur. Sie eignen sich zum Abtragen von Silikon, zum Abrunden von Kanten und zum feinen Glätten der Oberfläche. Max. Drehzahl 10.000 U/Min. Wegen der feinen Körnung nur mit geringem Druck arbeiten.

Mundschutz und Schutzbrille tragen!

Lisko-Polierscheiben sind mit Kunstharz getränkte Vliese und geringen Anteilen von Schleifkörnern. Sie sind weich, elastisch und somit ideal zum Beschleifen und Glätten von flexiblen Kunststoffen, Silikonem oder anderen weichbleibenden Materialien. Lisko-Polierscheiben eignen sich auch zum Vorpolieren von Metall und anderen Kunststoffen.

Die Polierscheiben passen sich flexibel den Konturen der zu bearbeitenden Oberfläche an. Sie zeichnen sich durch geringe Wärmeentwicklung aus, schmieren nicht und gewährleisten durch ständig frei werdende Schleifpartikel gleichbleibende Ergebnisse.

Lieferform:

413 678 Lisko-Polierscheiben fein, weiß
10 Stück, Ø 28 mm, unmontiert,
4 Stützscheiben aus Kunststoff und 1 Mandrell

Lisko-polishing discs, fine white



Lisko-polishing discs have an open structure. They are suited for removing silicone, for rounding edges and for fine smoothing of the surface. Max. speed 10.000 rev/min. Due to the fine grain size only work with low pressure.

Wear mask and safety goggles!

Lisko-polishing discs are fleeces soaked with synthetic resin and a small amount of ground parts. They are soft, elastic and therefore ideal for grinding and smoothing flexible plastics, silicones or other soft remaining materials. Lisko-polishing discs are also suited for prepolishing metal and other plastics.

The polishing discs flexibly adapt to the contours of the surface to be treated. They distinguish by poor heat development, do not smear and ensure consistent results due to constantly released grinding parts.

Availability:

413 678 Lisko-polishing discs, fine white
10 pieces, Ø 28 mm, unmounted,
4 supporting discs out of plastic and 1 mandrel



(Hersteller)

Erkodent Erich Kopp GmbH • Siemensstraße 3 • 72285 Pfalzgrafenweiler, Germany

Tel. + 49 (0) 74 45/85 01-0 • Fax + 49 (0) 74 45/85 01-15 • info@erkodent.com • www.erkodent.com

EN ISO 13485 • ISO 9001