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Onyclip-Set (270 500)
 Seitenschneider (270 502)
 Rundzange (270 503)
 Klebe-Gel (270 504)
 Entfetter (270 507)
 Onyclip-Streifen 0,10 mm (270 511)
 Onyclip-Streifen 0,15 mm (270 516)
 Schleifstein

Tipps

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Hints



Thoroughly degrease nail and clasp.
 Do not wait too long before applying.
 Only **press lightly** while glue is hardening.
 Hold Onyclip app. 1 min. in the correct position.
 Do **not let slip** while glue is hardening.
 Do not bend Onyclip strips.
 If too tight remove Onyclip with a flat spatula.
 Wait 12 hours before applying polish to Onyclip.
 All numbers in brackets are order numbers.
 Erkodent service: info@erkodent.com

1. Measure the width of the nail by using the enclosed measuring strip. To prevent irritations in the nailfold the correct length of the Onyclip strip corresponds to the width of the nail minus 1 - 2 mm.


2. Select the Onyclip strip matching the nail or corrective thickness. Light beige 0.1 mm, beige 0.15 mm thick.

3. Place the measuring strip along the selected Onyclip strip and cut it to size with the diagonal nipper.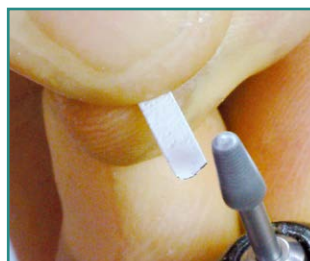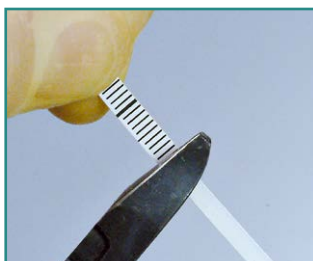



4. Round off and smooth cut edges with the abrasive stone without damaging the plastic coating.

5. In the adhesive areas left and right smooth the nail with the abrasive stone (hollows, grooves) to achieve a flat fullsurface adhesion.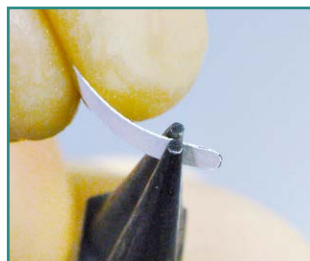

6. If necessary, bend the Onyclip strip with the round nose pliers to match the shape of the nail, taking the required spring force into consideration.

7. - 8. Remove any oil from nail and Onyclip strip with degreasing agent.

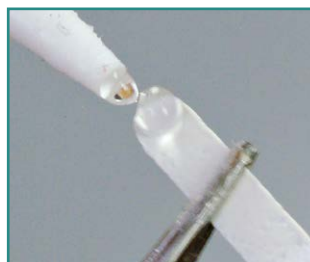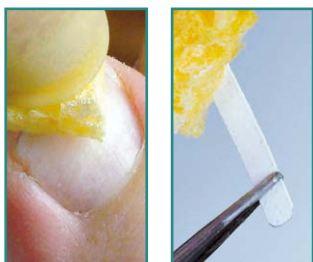9. Coat Onyclip strip on one side with one drop of adhesive gel.

10. Then lightly press on the Onyclip strip with the spatula approx. 1 minute in the right position with **light pressure** so as to obtain the largest adhesive contact between the nail and Onyclip. The clasp must not slip during the gluing procedure.

11. - 12. Wait app. 1 minute and follow the same procedure on the other side. To avoid any risk while gluing the Onyclip clasp with the spatula, the clasp can be fixed with a transparent adhesive strip. After app. 12 hours nail polish can be applied on Onyclip.