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Beiliegende Dokumente beachten

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EG-Richtlinie 93/42/EWG

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## Lagerung:

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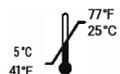
Trocken lagern

Empfohlene Lagertemperatur

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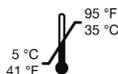
Recommended storage temperature

Recommended working temperature

Paste for the construction of correction orthosis by OSM Türk, two stage impression technique and pads in the nail groove. High flowing paste quick setting, easily mouldable, elastic, easy to grind, washable comfortable to wear and also combinable after the hardening.

### Application:

Mix **Sitran-OF** thoroughly with the determined quantity of hardener paste by a mixing spatula so as to create a mixture which should be of uniform colour with no streaking present. After exact mixing, the material should be evenly put on all sides of the tubular bandage, put into the nail bed groove or into the Kneton impression. The material has set after app. 1.5 – 2 min. The final hardness of app. 36 Shore A is reached after app. 1 hour.

**Sitran-OF** can be ground by abrasive belts, pumicers or multiple-discs. To polish the surface we recommend to use Lisko polishing discs fine, white (art. no. 413 678). In cases involving clavus or similar conditions no hole should be provided so that the humid milieu will encourage the healing process. Always use some footcream when adapting it to the patient.

### Dosage:

To one length of **Sitran-OF** (5 cm = ca. 5 g), half a length of hardener paste (2,5 cm = ca. 0,66 g) is added.

### Technical Data:

- mixing time 30 - 45 sec.
- total working time 1.5 - 2 min.
- setting time 6 - 8 min.
- app. 36 Shore A
- mixing can be carried out manually or by machine
- store **Sitran-OF** in a dry place at room temperature
- disinfection of the orthosis is possible
- washable

### Important:

Higher room temperatures and more hardener will accelerate the setting. Avoid eye contact with hardener; avoid stains on clothes.

The user of our products is the only person responsible for the use and the application of our products. Our verbal and written advice has to be considered as advice without obligation also regarding possible protection rights of third parties. The user is not exempt from the obligation to check our products personally concerning suitability for the intended application. The perfect quality of our products is guaranteed according to our General Terms and Conditions. In any case and for all damage our liability is limited to the value of the goods supplied by us.

 (manufacturer)  
 Erkodent Erich Kopp GmbH • Siemensstraße 3 • 72285 Pfalzgrafenweiler, Germany  
 Tel. + 49 (0) 74 45/85 01-0 • Fax + 49 (0) 74 45/85 01-15 • info@erkodent.com • www.erkodent.com  
**EN ISO • 13485:2016 • ISO 9001:2015**