

Duplication moulds

Materials & Accessories

Fabrication:

- Erkoflex, 3.0 - 5.0 mm

Model preparation:

- High-fusing wax (725 080) for filling bubbles in the plaster



Hints

- The duplication with Erkoflex is not as precise as the duplication with duplicating compounds. But this very low-cost method is suitable for demonstration, training and planning models.

1. Trim the model flat, put it onto the model plate in the thermoforming unit and thermoform.

Allow to completely cool down.



2. Put the formed Erkoflex as shown onto a ring or a pot in a way that the negative has contact only at the outside.

Put in plaster while form is standing on the vibrating slab.