23°F
-5°C

45°C
113°F

Erkodur-S

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EN: Instructions for use

Intended use: Thermoforming sheets for the manufacture of multi-layered protection splints (personal protective equipment).

Field of application: Erkodur-S is exclusively suitable for the production of hard inserts for three-layer protection splints made of Erkoflex.

Fabrication, see thermoforming technique brochure and videos, www.erkodent.com  

Contraindications: In case of already known allergic reaction to plastics a specific allergy to Erkodur-S (SBS) should be excluded.

Precautionary measures: Erkodur-S is recommended exclusively for the above mentioned appliances. Observe storage instructions.

Attention: The use is limited to the fabrication of three-layer protection splints made of Erkoflex. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Erkodur-S must be reported by the user to the competent authority and to the manufacturer.

Erkodur-S (Circa values): Density 1.02 g/cm³, modulus of elasticity 1900 MPa, elongation at break 20%, water absorption 0.2%, temperature resistance 55 °C

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