Erkodur-al/ Erkodur-al 240

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EN: Instructions for use
Erkodur-al (Ø 120 +125 mm) and Erkodur-al 240 (Ø 240 mm), colorless, clear
The general purpose of Erkodur-al is the manufacture of appliances for dental and orthodontic treatment.

Application (material)	Intended use	Indication	Patient groups
Aligner (Erkodur-al-/240)	Positioning of teeth	Misaligned teeth	Adults, teenagers
Retainer (Erkodur-al-/240)	Stabilization of tooth positions	Misaligned teeth	Adults, teenagers

Contraindications: If an allergic reaction to plastics is already known, a specific allergy to Erkodur-al (Copolyester) should be ruled out.
Caution: Sensitive to moisture, close the aluminum bag tightly again immediately. Observe storage symbols.
User: Dental professionals
Intended use: Erkodur-al is recommended as a medical device exclusively for the above-mentioned purposes.
Note: The use is subject to responsibility and is based on the recommendation/prescription of the dentist. Allergic reactions are unlikely, but cannot be ruled out. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small fractures.
Combination products: Erkodur-al can be processed in all devices of the Erkoform and Erkopress series from Erkodent.
Cleaning: Dental splints can be cleaned with Oxydens cleansing tablets. Also with curd soap, water and a toothbrush. Toothpaste is not suitable. Do not heat above 50 °C. Please rinse the splint with clear water before inserting it. As the user, please inform the patient about the cleaning options.
Further information: Thermoforming technique brochure and videos, www.erkodent.com
Serious incidents in connection with Erkodur-al must be reported. The user must inform the competent authority and the manufacturer.
Performance characteristics (Circa values): Density 1.19 g/cm³, modulus of elasticity 1462 MPa, elongation at break 179 %, water absorption 0.5 %, temp. resistance 85 °C

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