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EN: Instructions for use**Intended use:** Thermoforming sheets are indicated for the fabrication of dental and orthodontic appliances for the treatment of all patient target groups.**Field of application:** Erkodur is exclusively suitable for the fabrication of thermoformed intraoral splints and appliances such as occlusal splints, bruxism splints and stabilisation splints, for correction splints (aligner), dressing plates, cosmetic splints, Silensor® anti-snoring appliances and temporary appliances.Fabrication, see thermoforming technique brochure and videos, www.erkodent.com**Contraindications:** In case of already known allergic reaction to plastics a specific allergy to Erkodur (PETG) should be excluded.**Precautionary measures:** Erkodur is only recommended for the above mentioned appliances. Observe storage instructions.**Attention, sensitive to humidity:** Close the aluminium bag immediately.**Attention:** The use is limited to the production of dental and orthodontic appliances. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Erkodur must be reported by the user to the competent authority and to the manufacturer.

Erkodur (approx. values): density 1.27 g/cm³, modulus of elasticity 2200 MPa, elongation at break, 40%, water absorption 0.2%, temperature resistance 63 °C

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