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EN: Instructions for use

Intended use: Thermoforming sheets are indicated for the fabrication of dental and orthodontic appliances for the treatment of all patient target groups.

Field of application: Erkoflex-bleach is exclusively suitable for the fabrication of thermoformed intraoral splints and appliances such as bleaching and fluoride trays. Fabrication, see thermoforming technique brochure and videos, www.erkodent.com 

Contraindications: In case of already known allergic reaction to plastics a specific allergy to Erkoflex-bleach (EVA) should be excluded.

Precautionary measures: Erkoflex-bleach is only recommended for the above mentioned appliances. Observe storage instructions.

Attention: The use is limited to the production of dental and orthodontic appliances. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Erkoflex-bleach must be reported by the user to the competent authority and to the manufacturer.

Erkoflex-bleach (approx. values): density 0.93 g/cm³, modulus of elasticity 70 MPa, elongation at break 450 %, water absorption 0.2 %, temperature resistance 62 °C

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