## Erkoflex-82/-95 Sticks

45 °C  
113 °F

-5 °C

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### EN: Instructions for use

**Intended use:** Thermoplastic material is indicated for the fabrication of dental and orthodontic appliances for the treatment of all patient target groups.

**Field of application:** Erkoflex-82/-95 Sticks are only suitable as addition of thermoformed intra-oral splints like positioners, bracket transfer splints, soft splints, radiation protection splints, Playsafe triple sports mouthguard. Fabrication, see thermoforming technique brochure and videos, [www.erkodent.com](http://www.erkodent.com)

**Contraindications:** In case of already known allergic reaction to plastics a specific allergy to Erkoflex-82/-95 (EVA) should be excluded.

**Precautionary measures:** Erkoflex-82/-95 Sticks are only recommended for the above mentioned devices. Observe storage instructions.

**Attention:** The use is limited to the production of dental and orthodontic appliances. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Erkoflex-82/-95 sticks must be reported by the user to the competent authority and to the manufacturer.

Erkoflex-82 (approx. values): density 0.95 g/cm<sup>3</sup>, modulus of elasticity 20 MPa, elongation at break 850 %, water absorption 0.2 %, temperature resistance 54 °C

Erkoflex-95 (approx. values): density 0.93 g/cm<sup>3</sup>, modulus of elasticity 70 MPa, elongation at break 450 %, water absorption 0.2 %, temperature resistance 62 °C

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