Erkolen

23 °F
-5 °C
45 °C
113 °F

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EN: Instructions for use

Intended use: Thermoforming sheets are indicated for the fabrication of dental and orthodontic appliances for the treatment of all patient target groups.

Field of application: Erkolen is only suitable for the fabrication of thermoformed moulded pieces, as mould for temporary crowns and bridges, etching masks for bracket transfer and as spacer for fluoride gel.

Fabrication, see thermoforming technique brochure and videos, www.erkodent.com  

Contraindications: In case of already known allergic reaction to plastics a specific allergy to Erkolen (PE) should be excluded.

Precautionary measures: Erkolen is only recommended for the above mentioned devices. Observe storage instructions.

Attention: The use is limited to the production of dental and orthodontic appliances. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only (thermoforming). Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Erkolen must be reported by the user to the competent authority and to the manufacturer.

Erkolen (approx. values): density 0.92 g/cm³, modulus of elasticity 260 MPa, elongation at break 450 %, water absorption 0.1 %, temperature resistance 68 °C

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