Erkoloc-pro/240/blu, green, pink

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Erkoloc-pro, transparent, blu, green, pink (rund, Ø 120 +125 und quadratisch, 125 X 125 mm) Erkoloc-pro 240 (rund, Ø 240 mm)

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The general purpose of Erkoloc-pro is the manufacture of appliances for dental and orthodontic treatment.

Application (material)	Intended use	Indication	Patient groups
Aligner (Erkoloc-pro/240) - Medical Device	Positioning of teeth	Misaligned teeth	Adults, teenagers
Occlusal splints (Erkoloc-pro, all) - MD	CMD treatment	CMD	Adults, teenagers, children
Silensor-sl (Erkoloc-pro/blu, green, pink) - MD	Therapy of OSA and ronchopathy	OSA and ronchopathy	Adults, teenagers,
Fluoridation splints (Erkoloc-pro, ≙ 1,0 mm) - MD	Caries prevention	Caries prophylaxis	Adults, teenagers, children
Bleaching splints (Erkoloc-pro, ≙ 1,0 mm) - Cosmetics	whitened teeth	Miscolored teeth	Adults, teenagers,

Contraindications: If an allergic reaction to plastics is already known, a specific allergy to Erkoloc-pro (PETG/TPU) should be ruled out.

Caution: Sensitive to moisture, close the aluminum bag tightly again immediately. Observe storage symbols.

User: Dental professionals

Intended use: Erkoloc-pro is recommended exclusively for the above-mentioned purposes.

Note: The use is subject to responsibility and is based on the recommendation/prescription of the dentist. Allergic reactions are unlikely, but cannot be ruled out. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small fractures.

Combination products: Erkoloc-pro can be processed in all devices of the Erkoform and Ekopress series from Erkodent. Silensor-sl parts for Silensor-sl. Fluoridation splints tested with UltraEZ, Elmax gelee, Lunos Fluoridgel.

Cleaning: Dental splints can be cleaned with Oxydens cleansing tablets. Also with curd soap, water and a toothbrush. Toothpaste is not suitable. Do not heat above 50 °C. Please rinse the splint with clear water before inserting it. As the user, please inform the patient about the cleaning options.

Further information: Thermoforming technique brochure and videos, www.erkodent.com

Serious incidents in connection with Erkoloc-pro must be reported. The user must inform the competent authority and the manufacturer.

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