Occ3-PMF

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Occ3-PMF

Device for print models to use the Occluform-3 with Erkoform-3 units


Video
how to use
Occ3-PMF

Occ3-PMF, 188 575:

Model pot insert (1) with model disc (2) and clamping jaws (3), counterbite holder (4), and fixing key (5) for the clamping device.


The model insert (1) reduces the amount of air to be evacuated for faster and better molding.

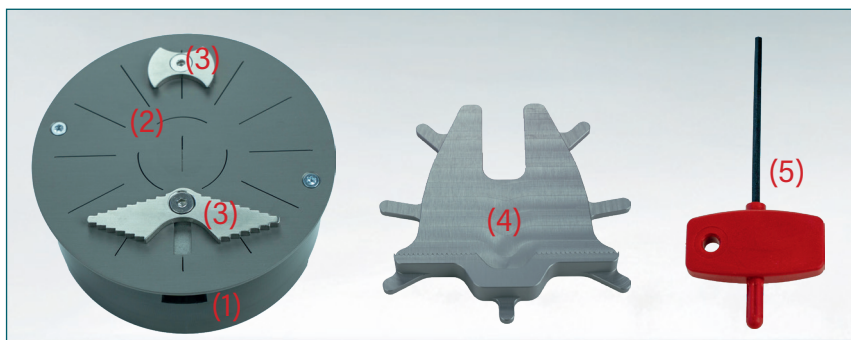
1. Take the Occluform-3 model pot out of the Erkoform-3 form pot. Remove possible impurities from the form pot.


3. Clamp the upper jaw model and fix the clamping jaws. The model must rest flush on the model disc.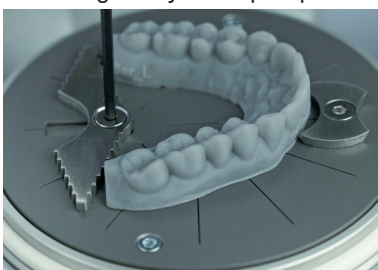

5. Place the counterbite model on the counterbite holder and fix with the Occluform-3 hex wrench.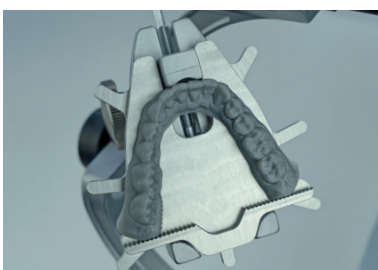

7. According to the Playsafe triple instructions clamp 2 Playsafe bite spacer over the counterbite model.

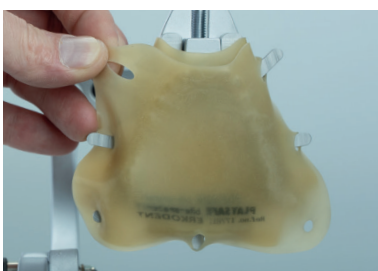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2. For it insert and engage the Occ3-PMF model pot insert. In doing so the wide clamping jaw points to the marking at the unit form pot.




4. Insert the Occ3-PMF counterbite holder in the Occluform-3 model plate.

6. Now articulate the models according to the Occluform-3 instructions (with construction bite, if available).

8. Continue and finish according to the Playsafe triple instructions.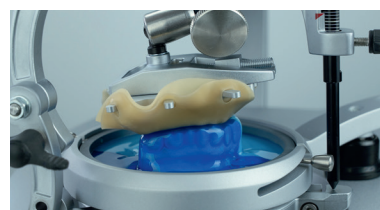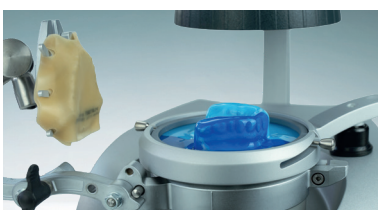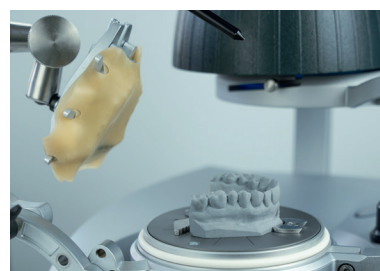

Playsafe triple
Fabrication

Tel.: + 49 (0) 74 45/85 01-0
info@erkodent.com
www.erkodent.com
EN ISO 13485 / ISO 9001

Occ3-PMF-anl-DE-EN-24-06-2025

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Erkodent Erich Kopp GmbH