
Silensor-sl Bauteile/parts

Page 1 of 1

Language: English

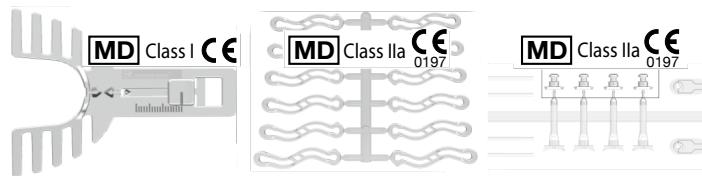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EN: Instructions for use

Intended use: Accessories for orthodontic and dental appliances for the treatment of all patient target groups.

Field of application: Accessories, consisting of connecting parts and an adjustable bite fork (sl-protrusion gauge) exclusively for the fabrication of the anti-snoring device Silensor-sl respectively Silent Nite.

Fabrication, see thermoforming technique brochure and videos, www.erkodent.com

Contraindications: In case of already known allergic reaction to plastics a specific allergy to polyamide (PA) should be excluded.

Precautionary measures: Silensor-sl parts are only recommended for the above mentioned devices. Observe storage instructions.

Attention: The use is limited to the production of dental and orthodontic appliances. Use is subject to the responsibility of the therapist. Use only on the recommendation or prescription of an authorised therapist. For single use only. Allergic reactions are unlikely, but cannot be excluded. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small breakages. Serious incidents in connection with Silensor-sl parts must be reported by the user to the competent authority and to the manufacturer.

(approx. values) Density 1,0 g/cm³, modulus of elasticity 1600 MPa, elongation at break > 50%, water absorption 3.0%, temperature resistance 115 °C

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