

# exMod-pro

Device for easy separation of  
thermoformed aligner foils from print models



exMod-pro, order no. **173 200**



Erkodent Erich Kopp GmbH  
Siemensstraße 3 • 72285 Pfalzgrafenweiler • Germany  
[info@erkodent.com](mailto:info@erkodent.com) • [www.erkodent.com](http://www.erkodent.com)



**exMod-pro**  
with mounted  
print model and  
thermoformed  
foil, ready for  
separation.

Suitable for all  
aligner foils.



Print model with  
hook-on pockets




Clamping template for foil  
thicknesses  $\leq 0.6$  mm and for  
particularly soft foils.


exMod-pro-info-EN-16-10-25



Hang the thermoformed  
foil and print model with  
the hook-on pockets onto  
the hook-on pins and  
press down gently.

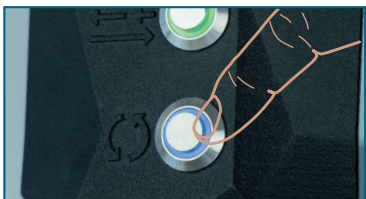

Press the blue illuminated  
switch to initiate the  
separation process.  
This process runs  
automatically.

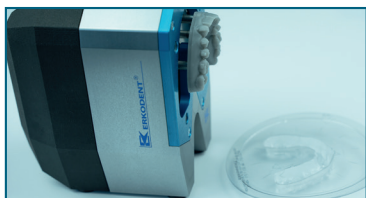

The foil is separated from  
the model and the model  
can be removed.