

# Helmet strap for American Football




**PLAYSAFE®**

+ **PLAYSAFE® triple**

Instructions:

- ▶ 1. Degrease the end of the helmet strap and the spot at the mouthguard where the strap shall be fixed (degreasing agent order no. 613 050).
2. Heat the spot at the mouthguard where the strap shall be fixed with the hot-air burner (order no. 177 540).
3. Heat the end of the helmet strap with the burner and ...
4. ... press it on the mouthguard immediately. Press bulges on flatly with the aid of a smooth, rounded instrument.

**ATTENTION!** These works must always be executed on the model, otherwise you risk to deform the mouthguard.

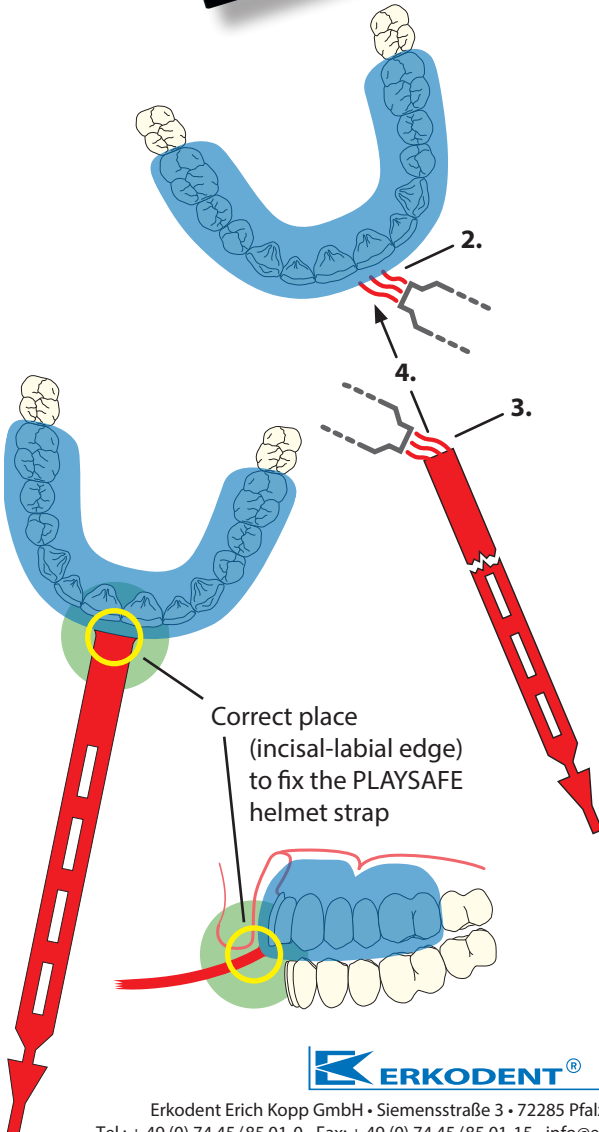

**ERKODENT®**